


# STRATTICE™

RECONSTRUCTIVE TISSUE MATRIX

PERFORATED

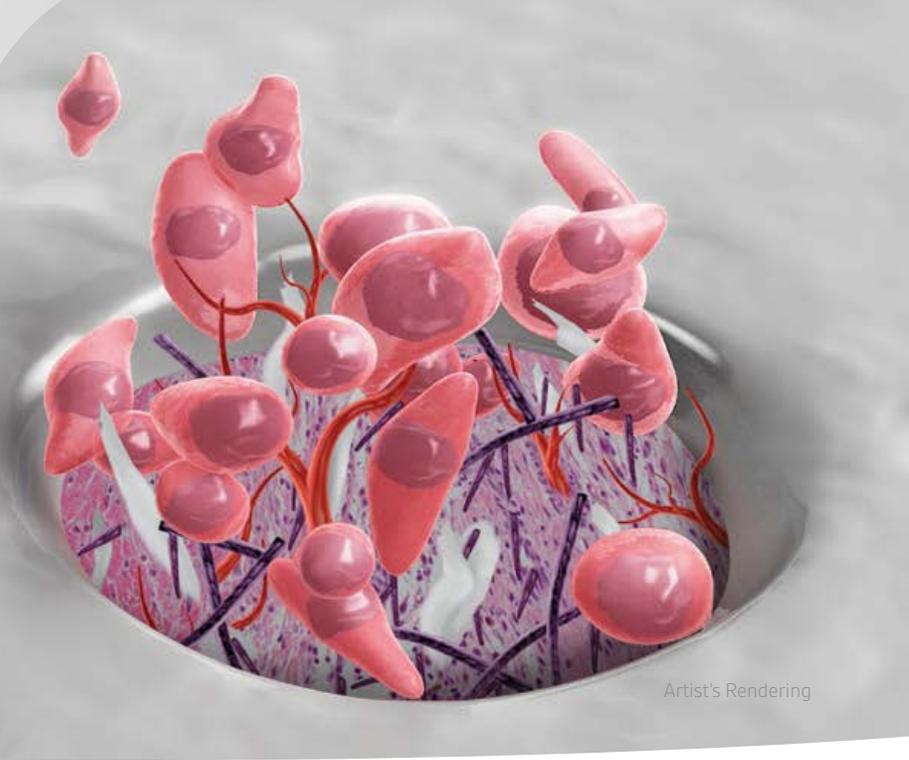

Designed to allow **fluid to flow** through the matrix upon implantation while supporting **regeneration** in complex abdominal wall **reconstruction**

STRATTICE™ Tissue Matrix Perforated combines the regenerative performance you have come to expect from STRATTICE™ Tissue Matrix, but is now perforated to help meet your surgical needs.



## Fluid communication

Designed to allow fluid to move through the matrix at time of implantation.



## Integration

Perforations facilitated tissue ingrowth as shown in a primate model.<sup>†\*</sup>




## Strength

Perforation pattern designed to maintain similar strength to STRATTICE™ Tissue Matrix.

## Out-of-Package Strength


Data representative of independent studies using same protocol\*

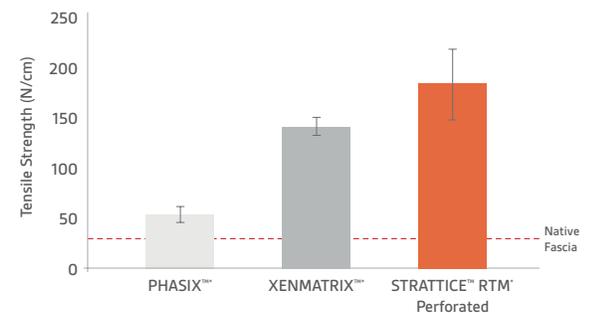

† Correlation of these results to results in humans has not been established.


\* Data on file; LifeCell Corporation

## Cell Repopulation: Perforations Facilitated Tissue Ingrowth as Demonstrated in a Primate Model<sup>†,\*</sup>

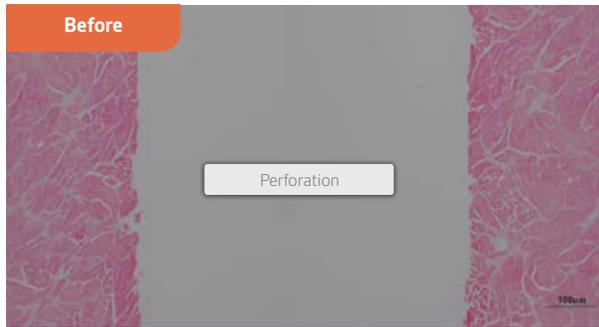


A post-hoc analysis revealed that cell repopulation more than **doubled** in areas adjacent to perforations, compared to areas within the central regions of the matrix.<sup>†,†</sup>

### Rapid Revascularization

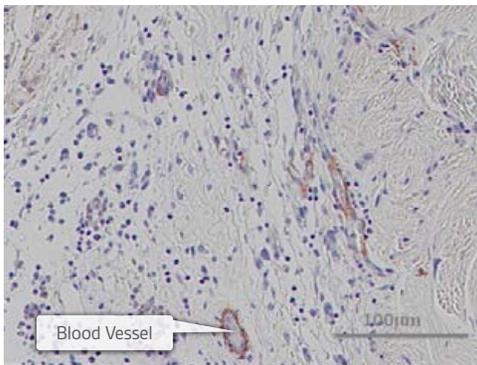


Smooth muscle  $\alpha$ -actin (SMA) staining demonstrated the structure of blood vessels, but only minimal myofibroblast cells in the perforation, suggesting a lack of scar formation.<sup>†,\*</sup>

### New Collagen Deposition

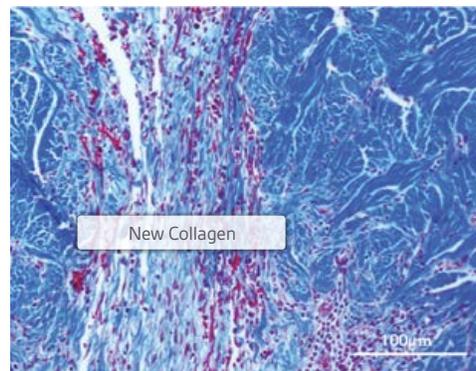

New collagen formation was demonstrated by Masson's Trichrome staining of the perforation.<sup>†,\*</sup>

### Minimal Inflammation in Perforations

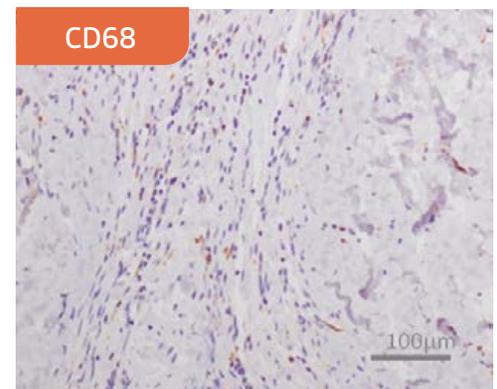

Minimal inflammation was detected in the perforations as shown by staining for macrophages (CD68).<sup>†,\*</sup>

<sup>†</sup>Correlation of these results to results in humans has not been established.

\* Data on file; LifeCell Corporation.

## Ordering Information

Product Size	Product Code	Coverage (sq cm)
10 x 16cm	1016002P	160
10 x 20cm	1020002P	200
16 x 20cm	1620002P	320
20 x 20cm	2020002P	400
20 x 25cm	2025002P	500
15 x 35cm	1535002P	525
20 x 30cm	2030002P	600
20 x 40cm	2040002P	800
25 x 40cm	2540002P	1000

Before use, physicians should review all risk information, which can be found in the *Instructions for Use* attached to the packaging of each STRATTICE™ Tissue Matrix Perforated. Rx only. CONTRAINDICATIONS: STRATTICE™ is derived from a porcine source and should not be used in patients with known sensitivity to porcine material, or in patients with a known sensitivity to Polysorbate 20.

For more information, contact your Lifecell Representative or visit [acelity.com](http://acelity.com)

LifeCell Corporation: One Millennium Way | Branchburg, NJ 08876 | Tel: 908-947-1100 | Fax: 908-947-1200  
Reimbursement Access Program, call 888-543-3656 or email [LifeCell@ReimbursementAccess.com](mailto:LifeCell@ReimbursementAccess.com)

©2016 LifeCell Corporation, an Acelity Company. All rights reserved. STRATTICE™ is a trademark of LifeCell Corporation. All other trademarks are the properties of their respective owners.  
MLC4544-R5/6092/9-2016